

SmartVision2

Release note – software version n° SV2.01.00.3746.USER

Software Update Process

An Internet connection is required to download and install SmartVision2 updates. To update your device, go to “Settings”, select "About Phone" and then "System Updates". Select "Download and install" at the bottom of the list and press the "OK" button to start the download.

Downloading the update may take some time depending on your connection. You can view its progress via the update notification in the notification pane. When the download is complete, select "Install" from the system update screen. It may take a few minutes to complete the process before you can use the product (vocalization feedback).

System improvements:

- **Overview:**
 - Bug fixes and performances improvements.
 - Consumption improvement
 - Translation improvement.
- **User Interface:**
 - Notification announcement improvement.
 - Improvement of accessibility of the virtual keyboard displayed on the screen
 - “Menu” button located on the title bar has been restored on 3rd party applications.
 - Improvement of keypad freezing.
 - Correction of the problem of keypad entry into internet page (web browser, google account setup, ...).
 - Automatic scroll for long text into list.
 - Task manager window has been improved (long press on “Home” key):
 - New design for a better accessibility to open or close an app,
 - Possibility to close all running apps from menu option.

Application Improvements:

- **Accessibility:**
 - Focus is no more stuck on the status bar on top of the screen.
 - Improvement of the vocalization interruption when browsing into list (download voices list, download maps list, gallery...)
 - Some features options are now correctly saved after a reboot (disabled screen On/Off, Disabled touch screen, ...)
 - Improvement of “Read from the top” feature from Global contextual menu and “# + 2” keypad shortcut.

- Sound feedback when setting vocalization output level by using keypad shortcut “# + volume Up / down”.
 - “Fast Boot” setting has been removed.
- **Audio book**
 - Select by format: PDF files are now correctly proposed.
- **Camera**
 - Icons and button from main screen are correctly vocalized.
- **Chrome**
 - Navigation by link (experimental – not available on all pages)
- **Colors detection**
 - New “Distance Assist” feature (experimental) allows to informed user when the Smartphone is too far from the target.
- **Contacts**
 - Improvement of the “Share contact” feature to Message or Gmail.
 - Improvement of vocalization of the contact detail.
 - Possibility to “Send a message” directly to the contact record screen.
- **Kapten**
 - “Download Maps”: possibility to search specific maps by using keypad input.
 - Improvement of the maps transfer from internal memory to SD card.
 - Last destination: street number is now correctly displayed and announced.
 - Ktag: street number is now correctly displayed and announced.
 - Itinerary: possibility to modify the starting point.
 - Direct Guidance: remaining distance is now correctly updated and announced.
 - Even if your current location is far from the road, the journey can start.
 - Select City: improvement of the cities selection.
 - Improvement of the Track feature.
 - Ktag: Voice memo feature has been removed.
 - Improvement of the Maps discovery feature.
 - During navigation, items from the menu are now correctly vocalized.
 - Status of the Free navigation activation is now correctly vocalized.
 - Correction of the accessibility issue to enter a street number.
- **Messaging:**
 - Possibility to start call the sender of the SMS.
 - Possibility to call or send a SMS to a phone number included into a message.
- **Magnifier**
 - Improvement of LED management when zoom level is changing.
- **NFC:**

- Correction of the problem to set a NFC tag with ringtone profile (Silent, normal, high, vibrate).
- **OCR:**
 - Improvement of the horizontality assistance announcement.
 - New “Distance Assist” option (experimental) allows to informed user when the Smartphone is too far from the target.
- **Phone:**
 - During a call, “Pause” button is now correctly vocalized.
 - Improvement of contacts frequently used layout.
 - List of “Access Point Names” has been updated.

Known bugs/ Corrections in progress:

- **User Interface:**
 - Settings: sometimes when you change the language of the product the voice stays on the previous language. Proceed with a second attempt.
 - FM Radio: impossible to record a radio when “Stop on exit” option is enabled.
 - Camera: some sub-menus and lists from settings are not accessible with keypad