

MiniVision

Release note – software version n° MV_6312

Software Update Process

An Internet connection is required to download and install MiniVision updates. To update your device, go to “Settings”, select "About Phone" and then "System Updates". Select "Download and install" and press the "OK" button to start the download.

Depending on your connection, downloading the update may take some time. When the download is complete, select "Install" from the system update screen. It may take a few minutes to complete the process before you could use the product (vocalized feedback).

System improvements:

- Overview:

- Addition of new features and new applications
- Bugs fix and performance improvement
- Translation and vocalization improvement
- Display improvement by writing in 1 line instead of 2 lines for some information.

- System:

- Fixed problem of blocking keypad.
- Fixed problem of voice recognition language setting in Danish.
- Fixed problem of very long delay during the Startup after a factory reset.
- Fixed problem of installing Premium Voice in Czech and Danish languages.
- Bluetooth: connection status for paired devices is now announced (see Settings / Bluetooth / Paired Devices).
- Fixed problem of keypad backlight, now the keypad is lighted whatever the key pressed.
- Fixed problem of time and date updating display on the home screen.

New application and new features:

Voice Commands

New voice commands are now available from home screen by doing a long press on the “OK” button.

An Internet connection (WIFI, 3G or 4G) is needed to activate the following keywords:

- Pronounce “**Keywords**” to have the list of all the keywords currently enabled.
- Pronounce the “**Name**” of an application to launch the application.
Example, say “*Phone*” to open the Phone application.
- Pronounce “**Call**” followed by the name of the contact: upon name confirmation, the call is made to the chosen contact.
Example: “*Call Paul Thomas*”.

Remark: It is equally possible to pronounce Thomas Paul or Paul or Thomas.

- Pronounce "**Send Message to**" followed by the name of the contact: upon name confirmation, a message is created for the chosen contact.
Example: "*Send Message to Paul Thomas*".

Remark: It is equally possible to pronounce Thomas Paul or Paul or Thomas.

- Pronounce "**Create Note**" followed by the text of the note: a note is created.
Example "*Create note, remember to wish a happy birthday to Suzan*"
- Pronounce "**New Alarm**" to open a new alarm page.
- Pronounce "**New Event**" to open a new event page from Calendar application.
- Pronounce "**New Contact**" to open a new contact page from Contact application.

Notes

The Notes application allows you to create, edit, delete and read notes in a text format.

Create a note

Press the "Menu" key and select "New Note" to create a new note. Enter your text and save your note by pressing the "OK" key.

The saved note is now available to be read from the main application screen.

The notes are sorted in chronological order, the most recent being placed on the top of the list.

Read and Edit a note

From "Notes" main application screen, select the note you want to listen to. Press the "OK" button and select the note from the "Edit" list to listen to it.

Other options are also available as follows:

- Copy all: allows you to copy the contents of the note
- Delete: deletes the note
- Export: copies the note into the memory of MiniVision.

Weather

This application allows you to have the weather forecast at your current location for the upcoming 5 days. You can also have a detailed forecast for each day. This application requires an Internet access (WIFI, 3G or 4G).

Flashlight

This application uses the backside LED as a flashlight. When the application is launched, the flashlight is switched on. Press the "OK" key or exit the application to switch the flashlight off.

Quick Access

This option is an improvement of the quick access dialing. Now, from the home screen, you can also associate and start an application with a long press on a numerical key (see also Settings / Keypad / Quick Access).

Application Improvements:

- **Accessibility:**

- New setting to change the text font (see into Settings / Display / Font)
- New setting to set font to bold (see into Settings / Display / Bold)
- New setting to show and vocalize password (see Settings / Security / Show passwords). By default, this setting is activated.
- New: when vocalization is disabled for Low-Vision people, the up and down keys will start the scroll of the text in an edit box.
- New: a beep is played when the beginning or the end of a list is reached.
- New: you are now prompted to “Speak after the beep” with a new pictogram on display when you start a voice recognition (voice command or dictation).
- Fixed problem of pressing “OK” key when the title of a pop-up was focused. Cancel action was generated.
- Fixed problem of non-systematically playing the validation beep sound when a character is input into an edit box.
- Fixed problem of symbols not available to set WIFI password.

- **Phone:**

- New: audio output level on the speaker has been improved during a call.
- New: press “Call pick-up” key to enable or disable the hand free mode (audio output on the loud speaker) during a call.
- Improvement of the management of phone settings like call forwarding, call waiting and Caller ID.
- Fixed problem of vocalization when a call is engaged now the call ID (phone number or contact name) is muted once the call is accepted.
- Fixed problem of vocalization during a call, sometime menu items wasn't vocalized.
- Fixed problem during incoming call, phone didn't vibrate when this option was enabled.

- **Contacts:**

- New: possibility import VCF file from SD card
- Fixed problem of wrong vocalization of the Contact name when calling a contact by using “Call pickup” key.
- Fixed problem of Contact with a short phone number which prevented an outgoing call.

- **Message:**

- New: possibility to send a message or call a contact directly from a discussion thread without the necessity to open a message first.

- New: impossible to send an empty message.
- New: confirmation message when a thread discussion has been deleted.
- New smileys have been added.
- Fixed problem of not receiving an answer from the Network carrier when a message is unsent, which was blocking the sending of other new messages.
- Fixed problem of forwarding a message.
- Fixed problem of unknow Smiley when reading a message, now vocalization says “Smiley” instead of nothing.

- **Alarm:**

- Fixed problem of not being able to go to sleep mode after an alarm event is stopped.

- **SOS/ Emergency:**

- Fixed problem when no phone number was set for an Emergency call
- Name of application changed to “Emergency” in English to avoid voice recognition issues.