

## SmartVision2

### Release note – software version n° SV2.01.00.3860.USER

#### Software Update Process

An Internet connection is required to download and install SmartVision2 updates. To update your device, go to “Settings”, select "About Phone" and then "System Updates". Select "Download and install" at the bottom of the list and press the "OK" button to start the download.

Downloading the update may take some time depending on your connection. You can view its progress via the update notification in the notification pane. When the download is complete, select "Install" from the system update screen. It may take a few minutes to complete the process before you can use the product (vocalization feedback).

#### System improvements:

- **Overview:**
  - o Bug fixes and performances improvements.
  - o Translation improvement.
  - o Audio improvement

#### Application Improvements:

- **Accessibility:**
  - o Vibrate after character input is now correctly working.
  - o Beep after character input is now correctly working.
- **Audio book:**
  - o Reading improvement of Epub, TXT, DOC, HTML format.
  - o SmartVision2 doesn't switch to sleep mode anymore when reading.
- **Kapten :**
  - o Improvement of the Ktag edition.
  - o Current position returns now the right information.
  - o Improvement of the navigation: immediate instructions are now correctly vocalized.
- **OCR:**
  - o OCR languages is now set automatically according to the system language.
  - o Improvement of the assistance instructions
- **Phone:**
  - o Possibility to dial the “+” symbol with a long press on “0” key.

## Known bugs/ Corrections in progress:

### - **User Interface:**

- FM Radio: impossible to record a radio when “Stop on exit” option is enabled.
- Camera: some sub-menus and lists from settings are not accessible with keypad