

SmartVision2

Release note – software version n° SV2-4259

Software Update Process

An Internet connection is required to download and install SmartVision2 updates. To update your device, go to “Settings”, select "About Phone" and then "System Updates". Select "Download and install" at the bottom of the list and press the "OK" button to start the download.

Depending on your connection, downloading the update may take some time. You can view its progress via the update notification in the notification shade. When the download is complete, select "Install" from the system update screen. It may take a few minutes to complete the process before you could use the product (vocalized feedback).

System improvements:

- **Overview:**
 - Bug fixes and performances improvement
 - Translation and vocalization improvement
 - New languages: Spanish, Finnish, Polish
- **System**
 - Display improvement of several native Android screens
 - “App permission” notification now disabled by default on the notification shade.
 - Input method in Hebrew: possibility to switch from Hebrew to Latin characters using the star key on the physical keyboard
 - List of applications now correctly sorted by alphabetical order even with special characters.
 - Vocalization improvement at product start

Application Improvements:

- **Accessibility:**
 - Improvement of notifications vocalization
 - “Missed called” from Wake-up announcement now supported for German and Italian languages
 - Hint of edit box now repeated when you delete all characters
 - Vocalization improvement when you modify a text. The character at the left of the cursor is now always announced.
 - Quick settings: accessibility improvement of the audio profiles selection
 - Character validation: timer 0.2 sec has been added
 - New Keyboard shortcuts added:
 - Enable/Disable Color inversion mode
 - Enable/Disable Mobile data usage

- Switch audio profile
 - Switch echo on input mode
- **Book Reader:**
 - ZIP files now supported
- **Contact:**
 - Contacts can now be deleted from the sim card
 - Contact search improvement in contact list: filtering by the first letter of the family name or first name is now available
 - Vocalization improvement for keyboard contact search
- **Clock:**
 - New design of the main screen. Items are now in a list format.
 - Accessibility improvement for Alarm, Clock, Timer and Stopwatch
 - Alarm can now be stopped with a short press on the power key
- **Email:**
 - Focus no more lost on the search bar when you type a character.
 - Improvement of attached files downloading
 - Quick access to attached files via the “star” key.
 - Attached files coming from Kapsys Email app, in the following formats: Epub, Epub3, Doc, Docx, PDF, Txt and Html, are now automatically opened via Book Reader
- **GMAIL:**
 - In your inbox, you now have the possibility to select multiple emails with a long press on OK key
- **Home screen:**
 - Internet bookmarks can now be added on the home screen
- **Internet:**
 - Vocalization and accessibility improvement of websites edit boxes
- **Kapten:**
 - Free Navigation will not be cut anymore if speed is less than 15 km/h
 - Free navigation: when a Ktag is near your location, the name of the Ktag is now announced instead of the postal address.
- **Phone:**
 - Contacts display is now synchronized between the phone and contact apps
 - Display improvement of the quick dial window
 - Improvement of the feature “Add the phone number to an existing contact”
 - Ringtone level no more announced when call ends
- **SOS:**
 - Possibility to set the SOS button delay from 1 to 5 seconds.

- **Third party:**
 - Voxiweb: Vocalization improvement of the edit boxes
 - WhatsApp: “Voice recording” button now accessible with the keypad. Hold the “OK” button to record and release it to end your recording.

Known issue:

- Depending of Google apps updates, it can be possible that two “Clock” apps are displayed into the application list: one is the “Clock” apps from Kapsys and the other one is the “Clock” apps from Google. Only Android “Clock” application can be uninstalled.