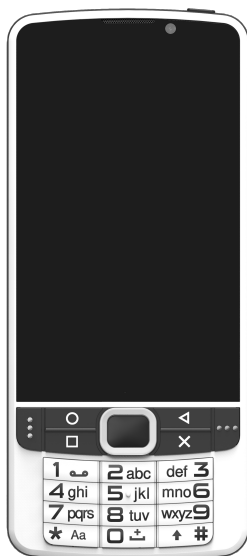
•SmartVision 2

Quick starting guide

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Guida rapida

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Hurtig guide

Hurtigstartguide

Snabbstartguide

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Ghid rapid de utilizare

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www.kapsys.com

Note

It is suggested to read the full user guide before using your SmartVision2.

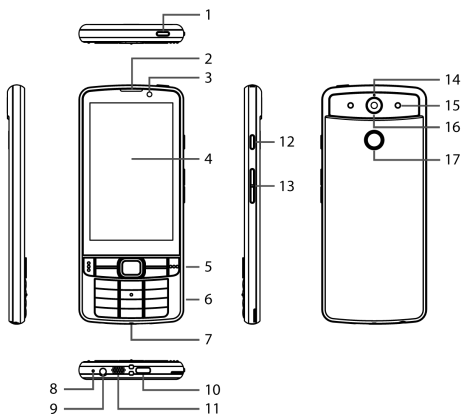
An electronic version of this document is available on our website at www.kapsys.com under the Support section. This document can also be consulted directly from your phone in the Home Screen / Menu / Help.

Box content

SmartVision2 is delivered with:

- A battery
- A 220/110 Volts mains charger – 5 Volts USB
- A USB cable (C type)
- A pair of stereo earphones with microphone
- A protection case
- A neck strap
- 3 NFC tags
- A Quick starting guide

Overview


N°	Item	Description	
1	Power button	Press and hold to turn the smartphone on/off. Press to turn the screen on/off (sleep mode)	
2	Smartphone receiver	Main receiver for phone calls	
3	Front-facing camera	Use for self-portrait images and online video chat	
4	Touchscreen	Display data, smartphone content and enter information	
5	Control keys and navigation pad		Navigate through the screen with multidirectional commands (up, down, right, left) and a central «OK» button
			Answer a call
			Hang up a call
			Go back to home screen
			Open menu associated to the current screen
			Go back to the previous screen
			Delete user input

6	Alphanumeric keypad	Use to input characters
7	Neck strap holes	Use to attach the neck strap
8	Microphone	Main microphone for phone calls and voice recognition
9	3.5 mm Headphone jack	Connect to stereo earphones
10	USB C type connector	Connect to a USB cable / charger
11	Loudspeaker	Main multimedia speaker
12	Voice command button	Use to dictate text and launch specific features
13	Volume buttons	increase or decrease smartphone volume
14	Second microphone	Used for noise cancelling
15	Flash LEDs	Enlightened for camera use, OCR, color detection, magnifier
16	Rear camera	8 Mega Pixels resolution
17	SOS button	Push to trigger the emergency procedure


Getting started

Insert the SIM card and the battery

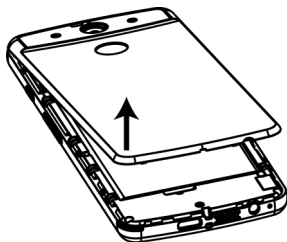
1. Turn the phone over, insert your fingernail into the notch located at the lower left corner and lift the back cover.

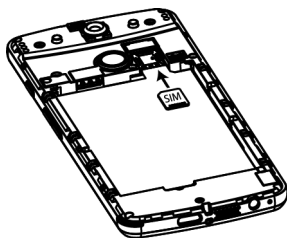
2. Insert the SIM card facing downwards and the cut-off corner placed at top left. Slide the SIM card upwards into the housing.

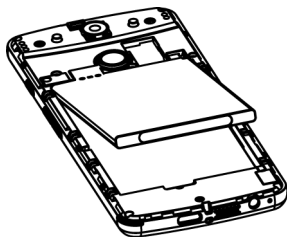3. Insert the battery by placing the upper section first, with the connection pads at the top left, then click the battery into place.

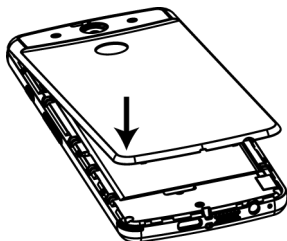4. Lastly, replace the back cover by aligning the tabs on the cover into the slots on the phone. Gently press the cover until it snaps in place.

Charging the battery

For initial use, it is recommended to charge the battery for at least four hours. After that you can recharge the battery as needed.

Connect the USB cable between the SmartVision2 and the AC charger. Plug the AC charger into any AC outlet and charge the device. You can also charge your smartphone by using the USB cable directly on your computer.

It is normal for the Smartphone to heat-up when charging or during long periods of use.

Turning your smartphone ON/OFF

Turn ON your SmartVision2 with a long press on the power button placed on the top of the phone. SmartVision2 vibrates when you successfully turn-it on.

When the phone is turned on for the first time, please follow the setup instructions (Language selection, profile selection, etc.)

SmartVision2 is ready to be used when the home screen is displayed.

When your SmartVision2 is ON, a short key press of the power button will bring it to standby mode (screen off), or back to running mode (screen on). The standby mode is activated after an inactive period for battery safe.

In standby mode, SmartVision2 screen is off but its features remain enabled. You can still receive incoming calls, messages, emails, etc.

To turn OFF SmartVision2, long press the power button, then select "Power OFF" and press the "OK" button. Follow the instructions.

If the screen remains black or if your phone is no longer responding, you can force the phone to restart by simply removing and replacing the battery.

Legal notice and Warranty

1. Precautions for use and recommendations

- Handle your SmartVision2 and its accessories with care, do not drop or knock them
- Do not dismantle the product as this will result in the warranty being voided
- Clean the product using only a soft, slightly damp cloth. Do not use chemicals, detergents or abrasive products
- This product is not waterproof. Protect it from damp and splashes of liquids
- Never use the device near a source of heat, and avoid storing the device in hot, humid or corrosive places
- Do not use the product near magnetic fields
- Do not use your device if the screen is broken or cracked, as this could cause injury
- Do not use the product in potentially explosive areas
- Do not charge the product in temperatures below 0°C (32°F) or above 45°C (113°F)
- Do not use the product in health care facilities and ask to authorized personnel before using your phone near medical equipment
- Do not expose your product to extreme temperatures. (1) Operating temperature range is -10°C (14°F) to +55°C (131°F); (2) short-term storage temperature is -20°C (-4°F) to +60°C (140°F) and (3) long-term storage temperature is -10°C (14°F) to +25°C (77°F)
- Your product contains a built-in lithium-polymer battery. To increase the battery life, always charge it fully
- Prolonged use of the product may cause the product to heat
- For optimal performance of your device, we recommend that you switch it off from time to time and remove the battery.

2. SAFETY AND HEALTH

- Keep the product away from small children
- Keep your hands free if and when you drive: drive your car in a safe and responsible manner and observe traffic rules
- Protect your hearing: avoid prolonged exposure to high sound levels when using the earphones, loudspeakers or the receiver
- If you wear a pacemaker or live near a person who wears one, you may want to seek advice from your doctor or the manufacturer of the pacemaker regarding its compatibility with a wireless device
- Radio transmitting equipment may interfere with the safe and effective use of some medical equipment when not adequately protected

3. WARRANTY

In case of breakdown, please contact your dealer.

No warranty is granted on the product, regardless of the malfunction encountered, in the following cases:

- Normal wear and tear of the product and reduction in the battery charge holding capacity
- Damage resulting from improper use, exposure to humidity or liquid, or proximity to a heat source
- Cracked or broken devices, or devices with visible signs of shocks
- Failure to respect the precautions for use, accidents, negligence, misuse or use inconsistent with the instructions provided with the device, or commercial use of the product

- Short-circuits of the battery or use of the battery in another device than SmatVision2
- Use of accessories or connectors not supplied by the manufacturer.
- Damage resulting to an attempted repair done by someone not authorized by the manufacturer

It is recommended that you perform one or more backups of the data saved in your product. The dealer cannot be held responsible for the loss of this data because of a malfunction, repair or replacement of the product.

Any malfunctions related to network services and / or cellular system cannot be guaranteed. The phone network provider or the internet service provider will need to be contacted directly for assistance.

4. ABOUT SPECIFIC ENERGY ABSORPTION RATE*

Your SmartVision2 by KAPSYS is a radio transmitter/receiver. It was designed and manufactured to comply with the radiofrequency (RF) exposure thresholds recommended by the European Union.

This device meets the EU requirements (1999/519/EC) on the limitation of exposure of the general public to electromagnetic fields by way of health protection. The unit of measurement for European council's recommended limit for mobile telephones is the "Specific Absorption Rate" (SAR). This SAR limit is 2.0 W/kg averaged over 10 g of body tissue. This device meets the requirements of the International Commission on Non-Ionizing Radiation Protection (ICNIRP) and European Standard EN 62209-1 / EN 62209-2 / NF EN 50566 for mobile radio devices.

For body-worm operation, this phone has been tested use with accessories that contain no metal and that position the handset a minimum of 1.5 cm from the body. Use of other accessories may not ensure compliance with RF exposure. If you do not use a body-worm accessory and are not holding the phone at the ear, position the handset a minimum of 1.5 cm from your body when phone is switched on.

During use, the actual SAR level is usually much lower than the maximum value. In general, the closer you are to the base station, the lower the transmission output of your mobile telephone. To reduce exposure to RF energy, use a hands-free accessory or other similar option to keep the device away from your head and your body.

The highest SAR value of this device is listed below:

- Head: 0.178 W/kg (averaged over 10g of body tissue)
- Body : 1.148 W/kg (averaged over 10g of body tissue)

5. CUSTOMER SERVICE

For more details, please contact your vendor's Customer Service or send us a mail to: Service Clients

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You can also visit our website: www.kapsys.com.

6. TRADEMARKS

Company and product brands referred to in this document and in the quick user guide, are trademarks, either registered or not, and owned by the respective holder as follows:

Kapsys words, brands and logos are registered trademarks of Kapsys SAS.

Words, brands and logos such as Android, Google keyboard, Google reader, Play Store, Google Play Music, Google Parameters are registered trademarks of Google Inc. in the United States and/or other countries.

ABBYY names, brands and logos are registered trademarks of ABBYY Software Ltd.

Bluetooth names, brands and logos are registered trademarks of Bluetooth SIG Inc.

Nuance is a registered trademark of Nuance Corporation Inc.

All brand names and product names are trade names or registered trademarks of the respective companies.

7. ABOUT THIS DOCUMENT

Information published in this document is liable to change without notice. KAPSYS reserves the right to amend the contents of this document with no obligation to notify any person or entity whatsoever thereof. Kapsys cannot be held liable in the event of any technical or publishing errors or omissions contained herein, or in the event of accidental or indirect loss or damage resulting from the product operation or from using this document. KAPSYS endeavors to continuously improve our product and its features quality and performance. Kapsys therefore invites you to visit their website (www.kapsys.com) for the latest update on the use and operation of this product.

RoHS COMPLIANCE:

This product is in compliance with Directive 2011/65/EU of European Parliament and of the Council of 8 June 2011, on the restriction of the use of certain hazardous substances (RoHS) in electrical and electronic equipment and its amendments.

**WEEE DIRECTIVE**

This symbol on the product or on its packaging indicates that this product must not be disposed of with your other household waste. In accordance with European directive 2002/96/EC on waste electric and electronic equipment (WEEE), this electrical product must not under any circumstances be discarded as unsorted municipal waste. Please dispose of this product by returning it to its point of sale or to your council's local designated collection points for recycling.

**MARQUE CE**

This product complies with the CE marking requirements for use in residential, commercial or light industry installations, and satisfies the provisions of Directive R&TTE (1999/5/EC). For more information about CE marking, you can also visit our website www.kapsys.com.

CAUTION: Permanent hearing loss may occur if earphones or headphones are used at high volume for prolonged period of time.

***FCC Warning:**

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Any Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

The SAR limit of USA (FCC) is 1.6 W/kg averaged over one gram of tissue.

Device types SmartVision2 (FCC ID: 2AN8L-SMARTVISION2) has also been tested against this SAR limit. The highest reported SAR values for head, body-worn accessory are 0.83 W/kg, 1.19 W/kg respectively. The Max simultaneous SAR is 1.32W/kg. This device was tested for typical body-worn operations with the back of the handset kept 10mm from the body. To maintain compliance with FCC RF exposure requirements, use accessories that maintain a 10mm separation distance between the user's body and the back of the handset. The use of belt clips, holsters and similar accessories should not contain metallic components in its assembly. The use of accessories that do not satisfy these requirements may not comply with FCC RF exposure requirements, and should be avoided.

The antenna(s) used for this transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Emergency call

If any emergency arises, dial 112/911 (or other emergency call number) for emergency help.

Note:

Due to the nature of cellular networking, the success of emergency call is not guaranteed.

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Device types SmartVision2 (FCC ID: 2AN8L-SMARTVISION2) has

also been tested against this SAR limit. The highest reported SAR values for head, body-worn accessory are 0.83 W/kg, 1.19 W/kg respectively. The Max simultaneous SAR is 1.32W/kg. This device was tested for typical body-worn operations with the back of the handset kept 10mm from the body. To maintain compliance with FCC RF exposure requirements, use accessories that maintain a 10mm separation distance between the user's body and the back of the handset. The use of belt clips, holsters and similar accessories should not contain metallic components in its assembly. The use of accessories that do not satisfy these requirements may not comply with FCC RF exposure requirements, and should be avoided.

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This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:
-Reorient or relocate the receiving antenna.

- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

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(ENGELSK) No warranty is granted on the product, regardless of the malfunction encountered, in the following cases:

- Normal wear of the product and reduced battery charging and storage capacity
- Damage resulting from improper use, moisture or liquids, proximity to or exposure to a heating source
- Device cracked or broken or showing visible signs of impact

- Failure to observe the precautions for use, accidents, negligence, misuse or use not in accordance with the instructions supplied with the appliance or commercial use of the product
- Battery shorting or battery used in a device other than your product
- Usage of accessories or connectors not recommended by the manufacturer
- Damage caused by repairs or attempted repairs by persons not authorized by the manufacturer

Det anbefales at tage regelmæssigt backup af dine data. (ENGELSK)
The dealer cannot be held responsible for the loss of this data because of a malfunction, repair or replacement of the product.

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Your SmartVision2 is a wireless transmitter/receiver. It has been designed and manufactured to comply with the radiofrequency (RF) exposure thresholds recommended by the European Union.

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- Head: 0.178 W/kg (averaged over 10g of body tissue)
- Body: 1.148 W/kg (averaged over 10g of body tissue)

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Device types SmartVision2 (FCC ID: 2AN8L-SMARTVISION2) has also been tested against this SAR limit. The highest reported SAR values for head, body-worn accessory are 0.83 W/kg, 1.19 W/kg respectively. The Max simultaneous SAR is 1.32W/kg. This device was tested for typical body-worn operations with the back of the handset kept 10mm from the body. To maintain compliance with FCC RF expo-

sure requirements, use accessories that maintain a 10mm separation distance between the user's body and the back of the handset. The use of belt clips, holsters and similar accessories should not contain metallic components in its assembly. The use of accessories that do not satisfy these requirements may not comply with FCC RF exposure requirements, and should be avoided.

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- Size : 152 x 66 x 10mm
- Weight : 150g
- Touchscreen : 4" LCD IPS

HARDWARE

- Processor Type : Quad Core 64-Bit @ 1.3 GHz
- Internal memory : RAM 2GB, Flash 16GB
- External memory : SD card supported up to 64GB
- Cellular connectivity: 4G LTE / 3G+ / 3G / 2G
- Wireless connectivity : Bluetooth 4.0 BLE / WLAN 802.11 a/b/g/n 5GHz
- Satellite receiver : GPS, Glonass with A-GPS support
- Sensors : accelerometer, proximity and light sensors, digital compass
- Camera Resolution : 2 Mpixel front/ 8 Mpixel rear
- SIM card format: 4FF
- Connectors : Audio jack 3.5 mm, USB type C
- Battery : 2700mAh Li-ion (Standby time : 350h - Talk time : 12h)
- Flash Leds : 2

DECLARATION DE CONFORMITE
Declaration of Conformity

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Name and signature or equivalent marking of authorized person

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KAPSYS | Parc Haute Technologie | 694 Avenue Maurice Donat | 06250 Mougins | France
t +33 4 92 28 88 88 | f +33 92 28 88 89 | www.kapsys.com

DECLARATION OF CONFORMITY

Reference No. : WTS17S1092268E
Applicant : KAPSYS SAS
Address : Parc Haute Technologie Lot 8, 694, Avenue Maurice
Donat, 06250 Mougins, France
Manufacturer : The same as above
Address : The same as above
Product : Smart Phone
Model(s) : SmartVision2, SmartConnect2, KAP12000
Ratings : Battery DC 3.6V, 2700mAh
DC 5V, 1.0A, charging from adapter
(Adapter Input: 100-240V~50/60Hz 0.2A)
Adapter : Manufacture: Shenzhen Dissinger Digita., LTD..
Model No.: D31-02051000

Test Standards: FCC PART15 SUBPART B: 2016

The above product has been tested by us with the listed standard and found in compliance with the Code of Federal Regulations, Title 47-Telecommunications, FCC Rules and Regulations Part 15, Subpart B and the measurement procedure according to ANSI C63.4:2014. The label must be complying with the (a) of the Section 15.19 labelling requirements.

Supplementary Information:

The product herewith has been successfully tested according to the Code of Federal Regulations, Title 47-Telecommunications. Test samples were in an engineering configuration. FCC Conformity Letter of Authorization for marketing will be required based on the final product. Specific test configurations and results are published by Waltek Services (Shenzhen) Co., Ltd.

The statement is based on a single evaluation of the sample of above mentioned product(s). It does not imply an assessment of the whole production.

Waltek Services (Shenzhen) Co., Ltd.

FCC Designation No.: CN1201 Test Firm Registration No.: 523476
Tel: +86-755-8355 1033 Fax: +86-755-8355 2400
Http://www.waltek.com.cn E-mail: CB@waltek.com.cn

KAP0765333