

MiniVision2

Release note – software version n° MV2_8607

Software Update Process

An Internet connection is required to download and install MiniVision2 updates. To update your device, go to “Settings”, select "About Phone" and then "System Updates". Select "Download and install" and press the "OK" button to start the download.

Depending on your connection, downloading the update may take some time. When the download is complete, select "Install" from the system update screen. It may take a few minutes to complete the process before you could use the product (vocalized feedback).

Improvements:

- **Overview:**

- Fix bugs and improve the performance.
- Improve translation and vocalization.
- SD card management improvement: when a SD card is present, all new pictures, voice memos and notes will be automatically stored on it.
- Possibility to manage more than 4 digits PIN code on a SIM card.
- Settings : “Reset APN” option is now available from Settings / network.
- Update of APN profiles.
- New settings to manage the keypad backlight (Automatic / Always On / Always Off).
- Shortcut # +1 announcing Phone status: the network status information is now announced earlier in the sentence.

- **Voice command:**

- New: possibility to use “Call” followed by a phone number.

- **Alarm**

- Empty alarms are not saved

- **Calendar**

- Empty events are not saved
- New: possibility to set an event without reminder (select reminder / none), in this case the event is created and listed, but will never ring.

- **Phone**

- New option within the Phone application: "Hide caller ID". This setting allows you to hide the name and phone number of the caller during an incoming call so that MiniVision2 does not vocalize them out loud.
- Before a call is made, it is now possible to switch directly to the hand-free mode by pressing the “Call” key.

- improvement of the format of the date for Call History in English language (US / UK).
- **Contacts**
 - Improvement of the import of Contacts stored on SIM.
 - Services contacts from SIM are not visible by default.
- **Messages**
 - Reading mode: vocalization improvement for a long text.

New app:

- **User Manual:**

This application gives direct access of the MiniVision2 user manual (when available in your language).

Several reading options are available:

- Browsing the user manual by chapter
- Navigating into the text (up / down key) by sentence, by paragraph or by chapter
- Pausing or resuming the reading by using OK key